



- THE CULPRIT OF THE SKIN AGING,
- DECREASED HYALURONIC ACID,
- DEHYDRATION

TECHNICAL SPECIFICATION	
Voltage	1 10V/240V 50Hz/60Hz
DA	12V/5A
Power	50W
Display	4.3 inches LCD touch screen
Multi-needle	3 needles, 4 needles, 5 needles, 9 needles
Mode	Low speed / High speed / 10-250 times/min
Advantage	Sing needle, Multi-needle, Vacuum, 3 in 1
Single needle caliber	0.25mm, Depth: 0.25~10mm
Multi-needle caliber	0.25mm, Depth:0.25~5mm
Vacuum FloW	25L/min Pressure: 90Kpa

CERTIFIED BY



ISO 13485



[www.codeesthetics.com](http://www.codeesthetics.com)

CODE AESTHETICS

CORPORATE OFFICE  
B2-34/A First Floor  
Yamuna Vihar Delhi-110053

CONTACT :  
+91 7053883467  
[Info@codeesthetics.com](mailto:Info@codeesthetics.com)




CODE  
AESTHETICS

MESO  
MESO SKINJECTOR



MESO SKINJECTOR



Water mesotherapy is to inject hyaluronic acid into skin. Hyaluronic acid contains moisture, 200-300 times as rich as one's body. So that it makes the skin smooth, soft, bright and hydrated after the injection. In order to keep the skin hydrated, ladies always choose varied skin care products. Actually, skin care products only hydrate moisture on the cutin. So that over smearing those products cannot deeply provide moisture for the skin. As we know, acne, wrinkle and inelasticity would appear if skin lacks moisture. Water mesotherapy will solves all the problems by reason of lacking moisture, such as sallow skin, color spot, wrinkle... And it will supply moisture for dermis. Therefore water mesotherapy will works well on whitening, moisturizing and renewing inelasticity.

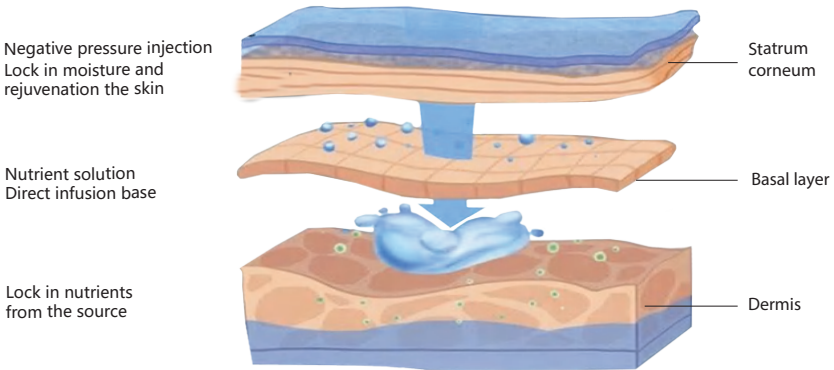

TREATMENT PRINCIPLE

Micro-needle mesotherapy gun using voice vacuum technology, accurately provides nutrients needed for skin in the depth of 1.28mm under the dermis. It is the principle of lifting the skin slightly by using sound pressure before micro-needle entered. The needle injects nutrients (such as hyaluronic and collagen) by using multi-needle sticking in the accurate depth after keeping skin tight. Injection rate can be easily adjusted through the button on OLED panel behind the device. Further more,the pressure of syringe had been liberalized before micro-needle was out,it will not bring the loss of nutrients.



- Magnetic Regimen
- Magnetic
- Stretch Marks
- Face Lifting
- Whitening Moisturizing
- Rejuvenation
- Germinal
- Vacuum Injection

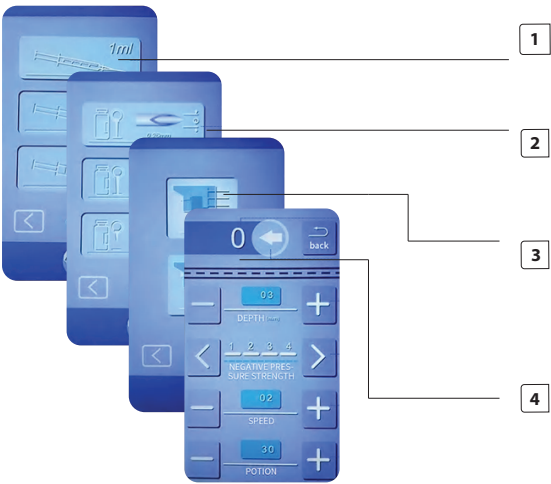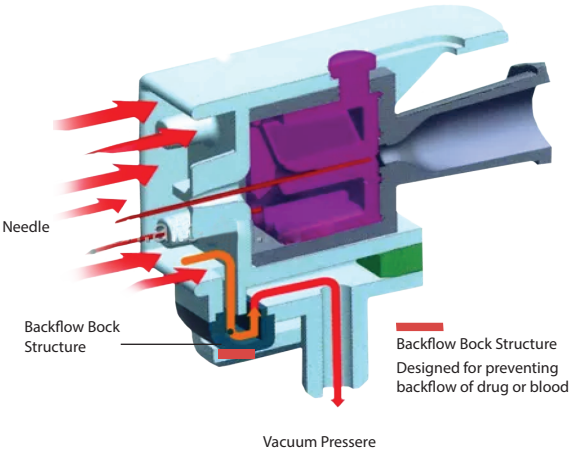

- STEP 1**  
Syringe specification interface:  
(Select the specification corresponding to the installed syringe)
- STEP 2**  
Injection Mouth diameter interface:  
(Select the caliber size according to the consistency of the injection)
- STEP 3**  
Needle Negative pressure interface:  
(It is generality recommended to choose negative pressure, and the negative pressere can be adjusted)
- STEP 4**  
Total parameter working interface:  
(Adjust different parameters according to the injected product)



- ❑ **ADVANTAGE**
- 1. Our handle is South Korea imported, as well as the needles.
  - 2. It has 5 pins injection technology which can speed up the treatment process.
  - 3. Adjustable injection depth, doses, speed and suction intensity.
  - 4. Anti-back flow technology is applicative for medicine and blood.
  - 5. Language: Chinese,English,Russian,French,German,Arabic.

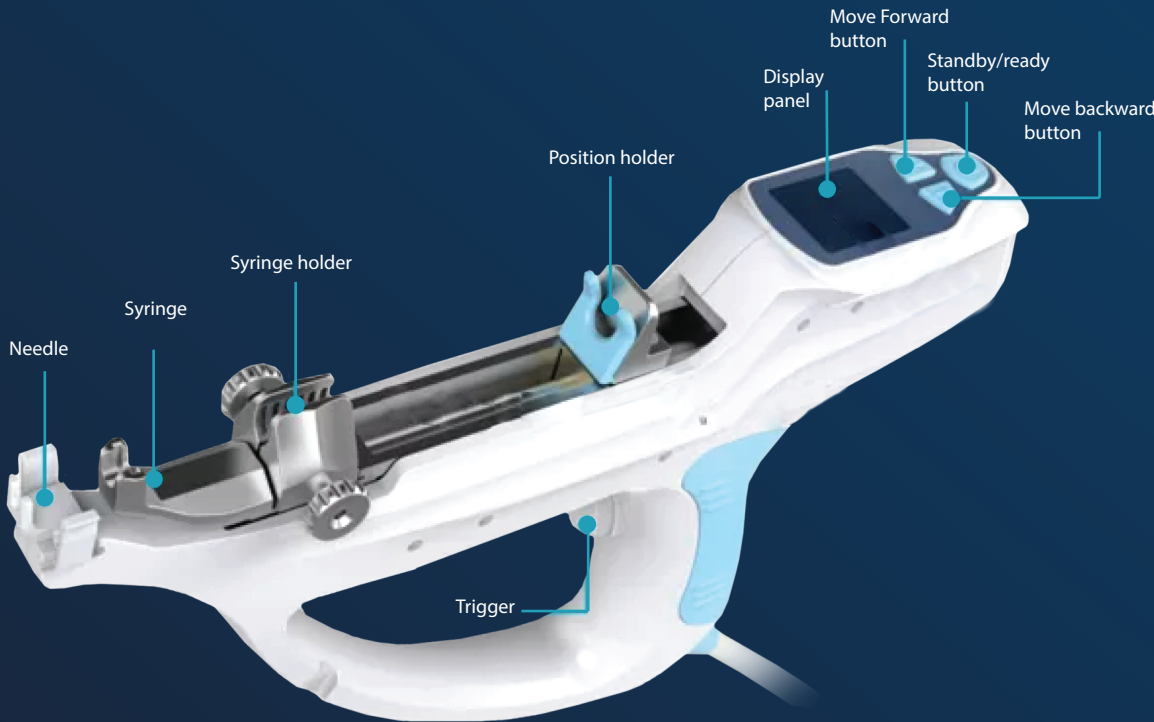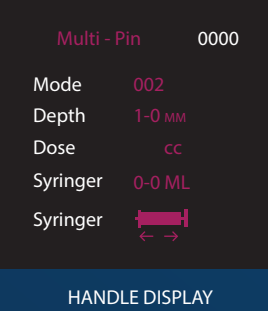

**THE EFFICACY OF NEEDLE MESOTHERAPY**  
Micro-needle mesotherapy use of the minimally invasive ways to stimulate bone colleagen, the skin without damaging the epidermis.

- 1 Using the principle of negative pressure adsorption, the nutritional factors are instantly injected into the skin to replenish nutrients
- 2 The molecular particles are smaller than the pore diameter and can penetrate deep into the basal layer
- 3 Choose the right nutrient solution for different skin problems
- 4 Nutrients go directly under the skin, hydrating and locking in water

